

**AFTER APPLYING SUBSTANCE TO NEW TEST STRIP,  
COMPARE RESULTS WITH COLOR INDICATIONS BELOW.**

**NEGATIVE RESULTS** —

**NEW STRIP**

**+ POSITIVE RESULTS**



**PART A** →

**PART B** →

**POSITIVE RESULTS:**  
Intense cherry red color

1. Remove test strip and swab from packaging.
2. Identify location of suspect residue. Insert swab into residue.
3. Rub swab through residue for 5-10 seconds and ensure sample is embedded in swab tip. Rub swab through Part A & B on strip for 5 seconds.
4. Positive color indications: Amphetamine - Intense cherry red color. Cannabis - Intense purple color. Cathinones: Soft red color.
5. Time for positive indication: Rapid, less than 2 seconds.
6. If no color indication observed after 10 seconds, suspect residue is unlikely to contain detectable narcotics.
7. If positive, immediately take photographic evidence of test strip. Document time, date, location on reverse of test strip. Initiate relevant security plan.
8. Note: Some of test zone reagents may be removed into swab tip during rub. This is NOT a positive reaction. Please check swab tip for positive color indication.